

bioMérieux COVID-19 AFIAS Combo, 20201028

BIO TECHNOLOGY
bioMérieux





AFIAS

COVID-19/Flu Ag Combo

Testing for COVID-19 and Influenza A/B

- Easy to use
- Automatic platform
- Room temperature storage
- Nasopharyngeal swab
- Connect to LIS/HIS
- Objective results

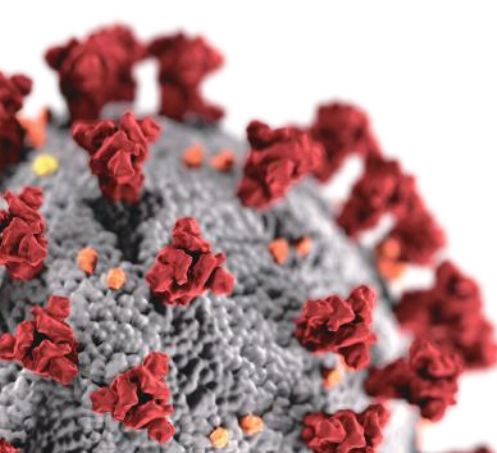

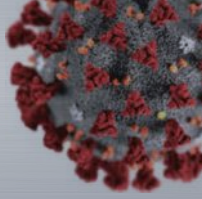


AFIAS COVID-19/Flu Ag Combo

- Health officials caution about the **twindemic** with COVID-19 and Influenza in coming flu season.
- **Both COVID-19 and influenza** can have common symptoms with various ranges from asymmetric to severe symptoms.
- **AFIAS COVID-19/Flu Ag Combo** can be diagnosed for distinguishing of COVID-19 and Influenza in patients with symptoms.


• Test procedure

3 simple steps




1 Insert the sterile swab used to collect the sample into the extraction tube and swirl 5 times

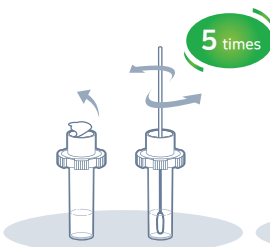

2 While squeezing the tube, pull the swab out

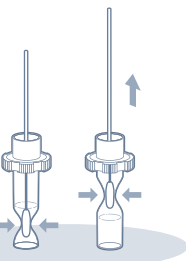

3 Assemble a nozzle and the extraction tube, and then fill the sample well completely with the sample mixture

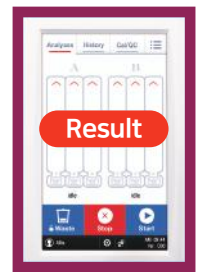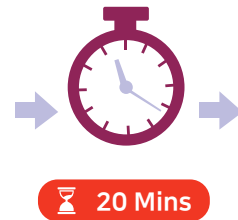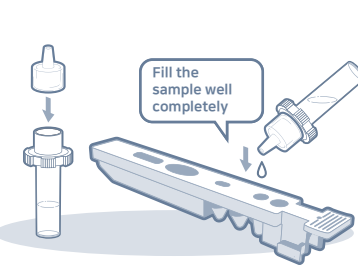

• Specification

Product name	AFIAS COVID-19/Flu Ag Combo
Assay type	TRFLFA*
Sample type	Nasopharyngeal swab
Sample prep / extraction	Collecting with nylon swab
Target	COVID-19 Ag / Influenza A Ag / Influenza B Ag
Platform	AFIAS 6 Reader / AFIAS 1 Reader
Unit	24 kit / box

* TRFLFA: Time-resolved fluorescent lateral flow assay



AFIAS COVID-19/Flu Ag Combo

• Ordering Information

Product	Cat. No.
AFIAS COVID-19 Ag / Flu Ag Combo	SMFP-77
AFIAS-6 Reader	FPRR020
AFIAS-1 Reader	FPRR019

* Note. A nylon swab is excluded from the components of the cartridge box.

