



How to use the DiaSpect Tm



Sampling



Disinfect and dry the puncture site



Make the incision on the side of the finger tip



Wipe away first 3 drops of blood then fill the cuvette without interruption



Wipe off the outside of the cuvette. Check that the cuvette is completely filled

Measuring



Insert the filled cuvette, press down gently and hold in position



The hemoglobin value will be displayed instantly



Pull out the cuvette quickly. Record the result as soon as the checkmark is shown



The result is displayed until the next measurement or until pressing down the cuvette holder

Cleaning

Use conventional solvent-free surface disinfectants, or alcohol based substances such as 70% isopropyl alcohol. Do not spray the instrument!



Pull the backside of the cuvette holder slightly towards you and lift up



Clean cuvette holder with cold water or mild detergent, followed by disinfectant



Reinsert the dry cuvette holder by pressing down until you feel a "click"



Clean device with cold water or mild detergent, followed by disinfectant